

Restoration success starts with an accurate matrix

Algin•X Ultra material lets you record the functional structure.

Predictable accuracy with dimensional stability

Algin•X Ultra alginate alternative delivers a high degree of accuracy, requiring less time adjusting. It creates a permanent record of functional structure, making it ideal for emergency provisional re-fabrication.

- High detail and 14-day dimensional stability for ease and convenience
- Easy automixing delivers a homogenous mix with no dust or cleanup
- Fast set time: 1'00" work time and 2'30" mouth removal time
- Pleasant flavor for patient acceptance

Tips for Success

- Always start by bleeding the cartridge
- Use a tray adhesive
- Don't dispense too much
- Verify proper maximum intercuspation
- Bur out holes on the buccal and lingual surfaces of the preliminary matrix to ensure easy removal of provisional from the mouth
- Take your impression with a dual arch impression tray like First Bite[®] disposable trays to ensure the patient is in occlusion, reducing checking and correcting time
- Keep the impression on file as a safety measure in case the patient breaks or loses their provisional



Ordering Information

- 61E800 **4-Pack Refill Cartridge - Fast Set**
4 Cartridges (50mL ea)
6 Mixing Tips
- 61E801 **Bulk Refill Cartridge - Fast Set**
24 Cartridges (50mL ea)
- 61E810 **DECA™ Standard Pack - Fast Set**
2 Cartridges (380mL ea)
20 Dynamic Mixing Tips
1 Bayonet Locking Ring
- 61E810-3 **DECA™ Economy Pack - Fast Set**
3 Standard Packs

