

# Accuracy and strength in every provisional.


Integrity Multi•Cure material lets you obtain critical fit, form, and function.

Repairs and remakes of provisionals can cost as much as \$900 or more a month in lost production<sup>1</sup>

## Delivers strength exactly when and where you need it

Integrity Multi•Cure material allows for immediate trimming and adjusting via on-demand light curing. Getting high strength sooner allows for faster bonding and procedure times than leading competitive products.


- Mouth removal time as early as 1'30" trim; & adjust 20" later via light cure
- Highly polishable to a natural luster for maximum gingival health
- Excellent esthetics: available in 7 shades with added fluorescence
- Unique dual cure formula

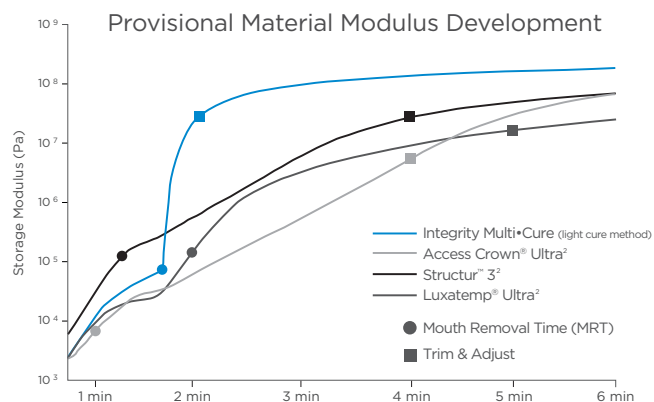

Note: ProTemp™ Plus² (multiple lots) failed cure profile testing.

## Tips for Success

- Always bleed the cartridge twice - before and after onset of the mixing tip
- Don't overfill - fill approximately 2/3 of the tooth in the matrix impression
- Set for 90" in the mouth, remove and light-cure for 20" before removing the provisional from the matrix
- Adjust and trim where needed right after light curing

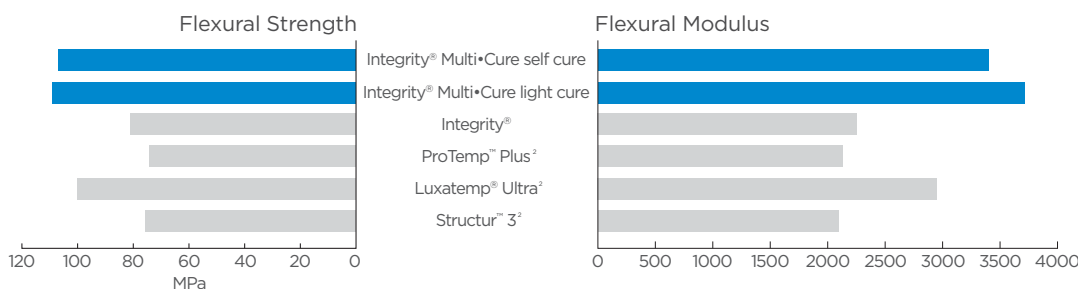

1. Data on file: 2013 Levin Group Annual Practice Research Report. Dental Economics, November 2013  
2. Access Crown Ultra, Structur 3, Luxatemp Ultra, and ProTemp Plus are not registered trademarks of DENTSPLY

# Ordering Information

## 666650 **Introductory Kit**

- 1 Cartridge Material - Shade A2 (76g)
- 15 Mixing Tips for Integrity Multi•Cure Material
- 2 Syringes Integrity® TempGrip® Cement (9g each)
- 20 Mixing Tips for Integrity® TempGrip® Cement
- 1 Cartridge Dispenser (10:1)

## **Cartridge Refills**

- 1 Cartridge Material (76g); 15 Mixing Tips

- 666600 Shade A1
- 666610 Shade A2
- 666670 Shade A3
- 666620 Shade A3.5
- 666630 Shade B1
- 666640 Shade BW
- 666680 Shade C2

- 666280 **Cartridge Mixing Tips Refill Package (20)**

