

# The right provisional cement sets the tone.

Integrity TempGrip cement lets you ensure an ideal environment for definitive cementation.

One single recementation visit can cost \$115 in office revenue<sup>1</sup>

## Minimize sensitivity and leave a clean field

Integrity TempGrip temporary crown and bridge material delivers strong cementation, smooth removal, and simple cleanup. It stays in the crown, not on the tooth, when you remove your provisional, leaving a clean field for easier definitive cementation.

- Less desiccation minimizes the threat of post-op sensitivity
- Material comes off smoothly without crumbling for quick cleanup
- Automix syringes with balanced base/catalyst viscosity ensure the right mix
- Low film thickness for superior luting along with high compressive and flexural strength provides reliable strength and security

	Flow (mm)	Work Time	Set Time	Film Thickness (microns)	Compressive Strength (MPa)	Flexural Strength (MPa)
Integrity® TempGrip®	27	1-1'30"	2-3'00"	11	22	9
TempBond® <sup>2</sup>	33	>1'30"	<7'00"	13	10	1
Zone (automix)	23	2'00"	2'30"	29	7	4
Zone (handmix)	22	2'00"	2'30"	37	6	2
TempBond® NE™ <sup>2</sup>	32	>1'30"	<7'00"	8	6	3
TempoCem® NE <sup>2</sup>	25	-1'00"	4-5'00"	24	5	4
TempoCem® <sup>2</sup>	34	-1'00"	4'00"	9	12	2
TempBond® Clear™ <sup>2</sup>	36	>1'30"	<5'30"	17	147	2

## Tips for Success

- Always bleed the cartridge twice – before and after onset of the mixing tip
- Do not desiccate the tooth before seating
- Remove excess cement quickly and easily with an explorer and dental floss



1. 2013 Levin Group Annual Practice Research Report. Dental Economics, November 2013

2. TempBond, TempBond NE, TempoCem NE, TempoCem, TempBond Clear, and Zone are not registered trademarks of DENTSPLY

# Ordering Information

666450 **Syringe Refill Package**

2 Syringes (9g ea)  
20 Mixing Tips

666460 **Mixing Tips Refill Package (50)**

